

PURISMA



Purisma+

Ultimate Computerised Multi-purpose Plasma
Lifting Platform & Trans Dermal Delivery
System TDDS For Professionals

Experience multiple treatments with four different
applicators provided for specialised treatments



INCLUDED ON ARTG REGISTRY,
CLASS 2B MEDICAL DEVICE #339842



MM MEDICAL
AESTHETICS

Less Pain, Effective & Versatile

● SPOT Plasma Applicator

Thin needle for classic carbon dot plasma treatments, thick needles for non-dot no downtime plasma treatments, eyelid lifting, lip flip.

● LARGE SPOT JET Plasma Applicator

Plasma & skin lightening - skin surface purification, acne reduction, hair growth stimulation, fine line reduction, TDDA enhancement, skin cell regeneration.

● SONO Poration Applicator

Sonophoresis handle utilises ultrasonic energy to infuse products into the skin. Ultrasound used in cosmetic applications has Thermal, Mechanical & Biochemical effects. Its Thermal action has a stimulating effect on blood flow & activates cell metabolism.

● ELECTRO Poration Applicator

Non-needle mesotherapy handle helps to infuse serums like Hyaluronic acid & vitamins into the skin resulting in wrinkle & fine line appearance reduction, better skin hydration, helps treat acne, oily, & greasy skin, helps add a more youthful appearance to the face, & it also helps with rosacea. Electroporation Mesotherapy treatments can be customised to help solve a variety of skin concerns you may have.



**CLINICALLY
PROVEN**



**NON-INVASIVE
TREATMENT**



**MULTIPLE
TREATMENTS**



**TIME
SAVINGS**



**COST
SAVINGS**

Treatments


- + Fine line reduction
- + Skin rejuvenation
- + Skin tightening
- + Eyelid lifting
- + Scar & hypertrophic scar reduction
- + Stretch mark reduction
- + Syringoma removal
- + Skin tag removal
- + Antibacterial & acne treatment
- + TDDS (Sonophoresis & electroporation)
- + Skin lightening
- + Rejuvenation
- + Hair growth stimulation
- + Toning





Spot Applicators

THICK Needle

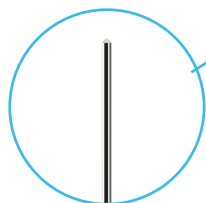

Used with thick gold needles designed for **No Downtime Plasma** treatment

Treatments


- Fine line reduction
- Skin rejuvenation
- Skin tightening
- Eyelid Blepharoplasty
- Lip flip
- Wrinkle reduction
- Scar reduction

Minimises pore size & surface irregularities resulting in tighter, smoother skin.

Usable Times

Needles can be autoclaved & reused

FINE Needle

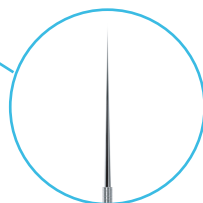

Used with thin FINE needles designed for **Classic Carbon Dot Plasma** treatment

Treatments


- Fine line reduction
- Scar & hypertropic scar reduction
- Stretch mark reduction
- Syringoma removal
- Skin tag removal
- Eyelid Blepharoplasty
- Lip flip
- Tattoo removal

Usable Times

Needles are disposable



Spot Modes

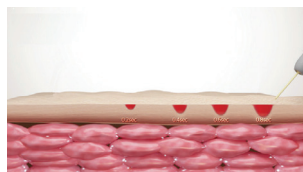

One-Shot Mode

The time of releasing plasma energy can be selected. As 'Time' gets longer, the total amount of energy transferred to the skin increases.



Pulse Mode

The plasma energy is released into a particular frequency during the time the button is pressed, & as the frequency increases, plasma energy is transferred to the deeper part of the skin.



Continuous Mode

The plasma energy continues to be released while the button is pressed. It provides a more convenient application for advanced therapists. It can also be used when removing scars of fibroma.

Large Spot JET Plasma Applicators

SQUARE Large Spot JET Plasma Tip



Treatments

- Antibacterial & acne treatment
- Skin lightening
- Rejuvenation
- Toning
- TDDS

SCALP Stimulation JET Plasma Tip




Treatments

- Hair growth stimulation




Large Spot JET Plasma Modes

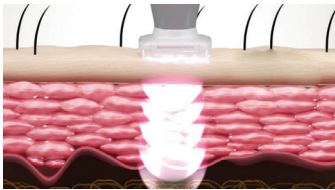

Pulse Mode


The time of releasing plasma energy can be selected. As 'Time' gets longer, the total amount of energy transferred to the skin increases.

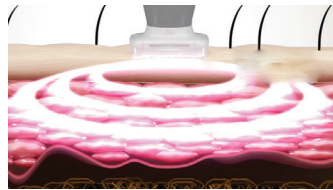

Continuous Mode

The time of releasing plasma energy can be selected. As 'Time' gets longer, the total amount of energy transferred to the skin increases.

SONO Poration

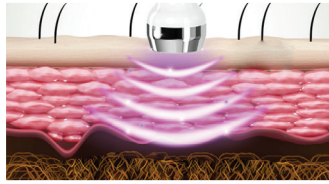

Features

Cell absorption & drug delivery with stable cavitation.

Stimulates the blood circulation & lymph system, strengthens skin elasticity, stimulates collagen regeneration.




Effect

TDDS
Dermal Wrinkles
Purification



ELECTRO Poration

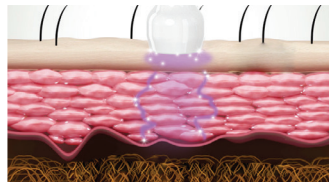

Features


Use of electric pulses, creating gaps in the stratum corneum layer of the skin so that beauty enhancers & medications can be introduced into the basal & dermis layers of the skin.

Collagen, hyaluronic acid, & other substances can be administered deep into the skin painlessly & quickly without having to rely on needles.



Effect

Anti-aging
Skin hydration
Lifting

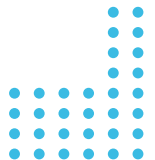

Before & After

Use of electric pulses, creating gaps in the stratum corneum layer of the skin so that beauty enhancers & medications can be introduced into the basal & dermis layers of the skin.



Eyebrow Tattoo Removal

Use of electric pulses, creating gaps in the stratum corneum layer of the skin so that beauty enhancers & medications can be introduced into the basal & dermis layers of the skin.



Before

Immediately after treatment



Before

6 weeks after treatment



Before

6 weeks after treatment


Plasma Skin Tightening

Use of electric pulses, creating gaps in the stratum corneum layer of the skin so that beauty enhancers & medications can be introduced into the basal & dermis layers of the skin.



Acne

Collagen, hyaluronic acid, & other substances can be administered deep into the skin painlessly & quickly without having to rely on needles.

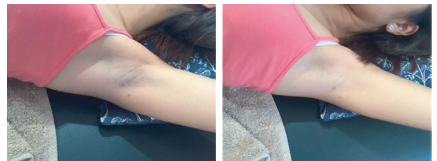

Underarm Toning

Collagen, hyaluronic acid, & other substances can be administered deep into the skin painlessly & quickly without having to rely on needles.

Specification

Dimensions

Main Unit	28(W) X 30(D) X 11(H) cm
Handpiece	24(W) X 4(D) X 5(H) cm

Weight

Main Unit	2.2kg
Handpiece	0.2kg

Power Input

100V - 240V / 60Hz

Maximum Output Power

SPOT	2.5W
JET	4.0W

SALES



-  (07) 5577 9881 | 0414 970 019
-  mike@beautyequipment.com.au
-  www.mmmedicalaesthetics.com.au

