



PROVALE CUP

Designed for people with dysphagia, The Provale Cup delivers a fixed measured dose of fluid.

When the cup is tilted in a typical drinking motion, liquid travels out of a measurement device. Before any additional liquid is released for consumption, the cup must be returned to an upright position and the drinking motion repeated. With each drinking action, the cup dispenses a small amount of liquid that many living with dysphagia may be able to safely consume.

Handles arrive detached and can be attached to one side or both sides.



No-Over
Delivery...
unlike a
"sippy" cup!

KEY FEATURES

- The Provale Cup, in a normal drinking motion, delivers 5 cc (1 Teaspoon) or 10 cc (2 Teaspoons).
- By returning the cup to its upright position, the chamber is refilled and ready to deliver another amount of liquid. This process encourages normal drinking habits and small swallows.
- The Provale Cup's ease of use may allow for more patient independence. Because individual patient needs vary, a trained healthcare professional must make the decision on patient independence.
- The Provale Cup has a patented feature that prevents the over-delivery of fluid when the cup is used properly.
- Sucking on the drinking spout will not result in additional fluid being dispensed.
- Proper Chin Position: The Cup-In-A-Cup design of The Provale Cup provides very good nose clearance which allows patients to keep their chin in a proper position to make a good swallow.
- Special Handles: The handles of The Provale Cup have been designed with patient needs in mind.

ITEM CODE

DL2005 Provale Cup 5cc Blue Top

DL2010 Provale Cup 10cc Brown Top



Quality
Approved
ISO 9001