

How to calibrate (add) new brand IV set on MP-60/SYS-6010

Step 1: Enter the Maintenance menu to activate "Maintenance" be "ON"

- 1). Press the [ON/OFF] key to turn on the machine. Press [HOME] to enter the main screen.
- 2). Click [OPTION] → [Maintenance] → [SN: xxx] as shown below.



- 3). Enter Password "1666" to enter Technical Maintenance screen.



- 4). Click [ON] key, then [Enter] key will be activated as shown below.



Step 2. Calibrate (add) new brands IV set

Take calibrating (adding) a B.Braun's 20d/ml infusion set of model 123 for example.

Preparation:

- 1) A new IV set to be customized, connecting it to an IV bag with more than 50ml liquid (pure water or saline solution).
- 2) A measuring cup graduation in ml (accurate to 1ml)

1. Edit IV set information.

1). Press [HOME] twice to return to main screen, click [INFUSION] → [IV set] → [NEW] to add new brand IV set information.



2). The screen will show as below after above operations. Click [Producer], [Model], [Size] one by one to enter the IV set information.





3). Click [NEXT]→[NEXT]→[CAL.] after editing [Producer], [Model] and [Size] to enter the screen shown as below.



2. Calibrate (add) the IV set

- 1). Load the IV set prepared to the pump. Prime the tubing manually.
- 2). Connect the measuring cup to the IV set.
- 3). Click [CAL.], and the screen shows "Calibrating".



- 4). After around 10 minutes, the calibration finishes automatically.



5). Measure the volume of the measuring cup.

6). Enter the volume obtained into the Volume box, click [OK]→[OK] to finish the calibration.

Note: Don't forget to return to the Maintenance screen to switch the Technical Maintenance off.

