**Wave** (Z220 with tub transfer base Z228)

riftonwave

## Wave A Bathing & Transfer System

**Hygiene and health go together.** When we make bathing easier, our clients are healthier. The Wave system has been thoroughly reimagined to improve bathing and showering for the client and for the caregiver.

Responding to requests from therapists and caregivers, we widened the seat to accommodate larger clients. And to protect skin integrity, we added generous padding under soft knit fabric.

In addition to the tub stand and rolling shower stand, we've added an ingenious tub transfer base to completely change the way you transfer and bathe large clients. There's nothing like it.



Wave bath chair is terrific! In the past, lifting a child into a tub has been a back killer, but the new transfer base eliminates this hazard entirely. The casters allowing lateral movement in the tub are a welcome addition, and the fact that I can easily collapse the base and hang it on the wall is a fabulous space saver.

Linda K. Miller, DPT Queensbury, New York

# **Wave features**



**Generously sized** The wide seat comfortably accommodates large clients. The seat width on the large size is 17½".



**One-hand adjustment** Adjust backrest angle with one hand, keeping the other hand on the client for safety and reassurance.



**Full leg support** The calf rest provides full support for the lower legs. Adjust it independently of the seat by pushing the gray buttons and rotating up or down. It must be installed at the time of purchase.




Typical bathtub height range

—18" —14"



## **G**rifton

### **WAVE** (BATHING & TRANSFER SYSTEM)

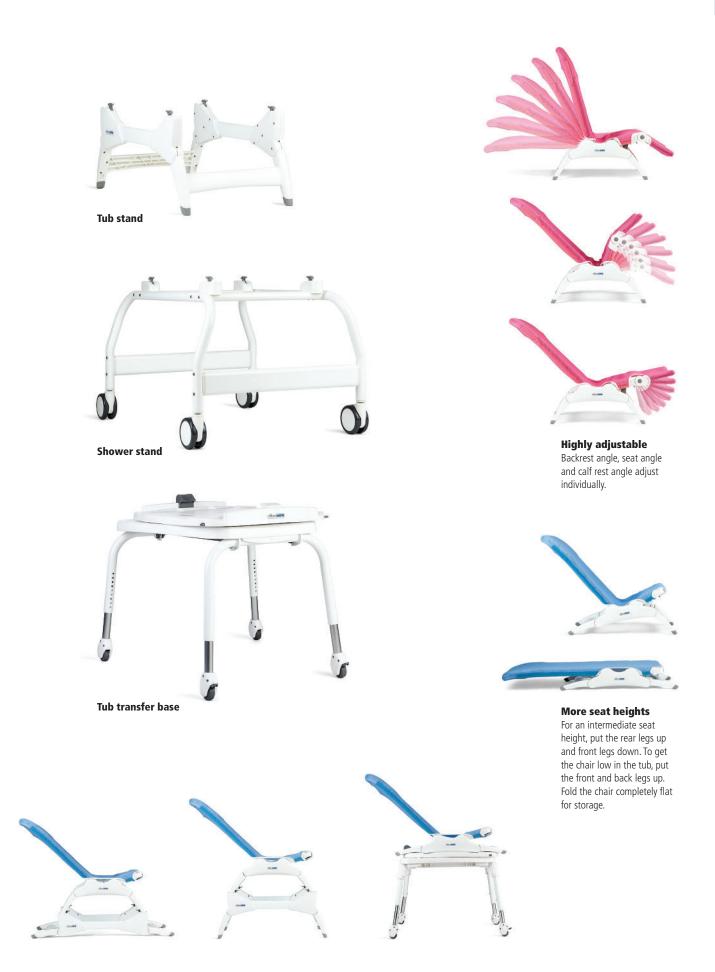

# Components

Bath chair size and color



#### **Choice of chest strap**



#### Without lateral positioning

The simple chest strap safely positions the user in the Wave bath chair. It attaches with hook and loop where needed along the full height of the back and buckles securely behind.



#### With lateral positioning

We designed this chest strap with adjustable lateral straps to give additional lateral support, helping users maintain better trunk alignment in the bath chair. The soft fabric laterals will not get in the way during transfers.





**G**rifton

### **WAVE** (BATHING & TRANSFER SYSTEM)



**Head blocks** Head blocks provide lateral support and

positioning for the user's head.

#### Leg straps

additional security).









#### **Calf rest**

The calf rest supports the lower legs. It is strongly recommended when the bath chair is used on one of the stands.

**Note:** The calf rest must be chosen at the time of the original purchase. It cannot be added later.





#### Tub stand

The tub stand raises the bath chair either  $5\frac{1}{2}$ " or 111/2" higher in the tub depending on the position of the tub stand legs.





#### Shower stand

The shower stand lets you roll your client into the shower stall at a convenient height for cleaning. Locking 4" casters provide smooth, easy maneuverability.



## Components (continued)

### **Tub transfer base**

Bathing equipment for people with disabilities tends to be big and bulky, taking up far too much space in bathrooms shared by the whole family. Using Rifton's tub transfer base, you can perform healthy transfers without crowding your bathroom with permanently mounted equipment. When bath time is over, the base can be folded up and hung compactly on the wall.



#### **Unique two-stage movement**



The tub transfer base starts in the position farthest out of the tub. It is then rolled back over the tub, and the brakes are set.



As the upper portion with the bath chair is rotated 90° it moves approximately 12" back over the tub in one motion.



The legs of the tub transfer base can be adjusted in increments of 5%" to fit bathtubs up to 24" high.



### **WAVE** (BATHING & TRANSFER SYSTEM)



1. With the client secured in the chair, push the transfer base to the back of the tub as far as it will go and lock the wheels.



**3. As the bath chair rotates,** it continues to move back over the tub. Gently lift the client's legs to clear the edge of the tub.



**2. Press either of the gray pivot latches** and rotate the bath chair and client over the tub.



**4. In the final position** the bath chair is over the tub and can be adjusted as desired for showering.

## Free up the tub.

**Rifton's folding,** storable tub transfer base makes it possible for everyone in the family to still use the tub.

When you are finished, remove the base from the tub, fold the legs and hang it on the wall hook provided.





# **Wave dimensions**

User dim	ensions (inches)	Z210 small	Z220 medium	Z230 large					
Height		up to 46	40-56	50-74					
	<b>Key user dimension: height</b> Select the appropriate bath chair by the user's overall height. Choose the size that allows for growth.								
	<b>Important:</b> Make sure that seat depth, width and back height are adequate. The user's weight must not exceed the maximum working load.								
Item dime	nsions (inches)	Z210 small	Z220 medium	Z230 large					
Seat depth		10½	13	17					
Seat width		15	15	17½					
Back height		23	26½	32					
Seat height		2–7	2–7	2–7					
Seat height (	incl. tub stand)	121⁄2-181⁄2	121/2-181/2	121/2-181/2					
Seat height (incl. shower stand)		271/2	271/2	27½					
Seat height (incl. tub transfer base) with long leg inserts		21-26 24½-29½	21-26 24½-29½	21-26 24½-29½					
Length of me	Length of mesh area when flat (without calf rest)		41	501/2					
Length of mesh area when flat (with calf rest)		421/2	49	61					
Base width	Base width		18	21					
Base length with legs flat	, ,	24 29½	24 29½	24 29½					
Tub stand he	ight (legs flat)	8	8	8					
Tub stand ler	ngth (legs flat)	39	39	39					
Tub stand wi	dth (legs flat)	17½	17½	17½					
Shower stan	nower stand height		22	22					
Shower stan	hower stand width		27	27					
Shower stan	ower stand length		34	34					
Tub transfer	base footprint	28 x 30	28 x 30	28 x 30					
Max. tub hei with long leg	ght for transfer base 9 inserts	20 24	20 24	20 24					
Maximum w	orking load (lb)	75	150	250					
Back and seat angle adjustments 0°, 15°, 30°, 45°, 60°,									



All three sizes are available in blue and pink.







# 2022 order form

Use dimension chart to select appropriate size.

### **Conversion kits**

The tool-free conversion kit converts your small chair to medium as your child grows.

#### Kit without calf rest

BlueZ268\$355PinkZ269\$355(Includes seat, back, cover and chest strap)

#### Kit with calf rest

Blue	Z266	\$440
Pink	Z267	\$440
(Includes seat, bac	k. cover and che	st strap)







			Z210 small		Z220 medium		Z230 large	
* Chair		Z211	\$290 🗌	Z221	\$300 🗌	Z231	\$380 🗌	
* Fabric cover	CH	Blue	Z244	\$215 🗌	Z246	\$225 🗌	Z248	\$235 🗌
	CHOOSE	Pink	Z245	\$215 🗌	Z247	\$225 🗌	Z249	\$235 🗌
* Chest strap	Œ	With lateral positioning	Z212	\$100 🗌	Z222	\$110 🗌	Z232	\$110 🗌
	CHOOSE	Without lateral positioning	Z213	\$90 🗌	Z223	\$100	Z233	\$100 🗌
Head blocks			Z214	\$100 🗌	Z214	\$100 🗌	Z234	\$100 🗌
Leg straps			Z215	\$90 🗌	Z225	\$100 🗌	Z235	\$110 🗌
Calf rest (Calf rest must be chosen at the time of the original purchase. It cannot be added later.)			Z216	\$130 🗌	Z226	\$147 🗌	Z236	\$168 🗌
Tub stand			Z217	\$300 🗌	Z217	\$300 🗌	Z217	\$300 🗌
Shower stand			Z218	\$800 🗌	Z218	\$800 🗌	Z218	\$800 🗌
Tub transfer base (calf rest recommended)	Œ	Counterclockwise into tub	Z227	\$950 🗌	Z227	\$950 🗌	Z227	\$950 🗌
	CHOOSE	Clockwise into tub	Z228	\$950 🗌	Z228	\$950 🗌	Z228	\$950 🗌
Legs for tub transfer base	CHOOSE	Standard	Z224	\$95 🗌	Z224	\$95 🗌	Z224	\$95 🗌
		Long (tub edge over 20" high)	Z238	\$95 🗌	Z238	\$95 🗌	Z238	\$95 🗌
Additional head blocks			Z214	\$100 🗌	Z214	\$100 🗌	Z234	\$100 🗌
Additional fabric cover			Z219	\$215 🗌	Z229	\$225 🗌	Z239	\$235 🗌
Additional chest strap		With lateral positioning	Z212	\$100 🗌	Z222	\$110 🗌	Z232	\$110 🗌
	CHOOSE	Without lateral positioning	Z213	\$90 🗌	Z223	\$100 🗌	Z233	\$100 🗌
Total			Z210	\$	Z220	\$	Z230	\$

\* Required