



Emergency Trolley - Accessories Installation



1: Push Handle

A: Allen Key



B: Push Handle



C: Screw (4 pcs)



Step A: Fix the push handle with four screws by allen key.

2: Accessory Bar

The accessory bar height can be adjusted





Step A: Loosen both screws a little (without removing) to allow bar height to be adjusted.

Step B: Once bar is at desired height tighten screws to keep it in place.

3. Defibrillator Shelf

A: Defibrillator Shelf



B: Knob



C: Installed



Step A: Insert the defibrillator shelf into the hole on the pillar and fasten the knob by hand.

4. IV Pole

A: IV Pole



B: Knob



C: Installed



Step A: Insert the IV pole into the hole on the pillar and fasten the knob by hand.

5. Needle Box

A: Needle box and holder



B: Allen key & screw



C: Installed



Step A: Put the needle box into the holder and fasten with strap.

Step B: Hang the holder to the accessory bar & fix the screw with the allen key.





6. Wire Basket

A: Wire basket



B: Accessory bar



C: Installed



Step A: Hang the wire basket to the accessory bar by hand.

7. Oxygen Tank Holder

A: Oxygen tank holder



B: Allen key & screw



C: Installed



Step A: Hang the oxygen tank holder to the accessory bar & fix the screw with the allen key

8. Power Board

A: Power board & holder



B: Allen key & screw



C: Installed



Step A: Hang the holder to the accessory bar, and fix the screw with the allen key.





9. Waste Bins

A: Waste bins



B: Waste bin holders



C: Installed




Step A: Hang the waste bins on the accessory bar by hand.

10. CPR Board

A: CPR board



B: Holder

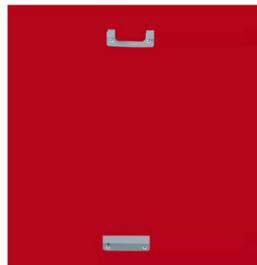

C: Installed



Step A: Hang the CPR board to the holder on the back of the trolley.

