

One Device  
**Infinite Possibilities**

Alma **Harmony**



# Alma **Harmony**

## XL PRO (SE)

### **YOUR ALL-IN-ONE CLINIC**

The Alma Harmony system is your one-stop-shop for a wide range of aesthetic needs, making it an exceptional addition to any clinic.

This powerful and versatile solution treats over 65 FDA-approved indications, maximising the potential of numerous energy-based technologies to achieve outstanding results.

Alma Harmony offers a full spectrum of technologies, applicators, tips, and treatment methods, enabling intelligent and patient-centric treatments that lead to enhanced outcomes.

As your clinic grows, the Alma Harmony platform can expand with you, with the addition of new applicators providing endless business opportunities. Compact and cost-efficient, Alma Harmony eliminates additional consumable costs, providing you with infinite possibilities.



**FDA approved for over 65 indications**

Alma Harmony XL Pro offers an all-in-one solution for many of the most common aesthetic concerns, for all skin types and ages.

**Extendable and upgradeable platform**

The upgradeable Alma Harmony XL PRO allows you to add extra functionality aligned with your clinic's growth strategy. No need for excessive device purchases – Alma is here to enhance your ROI.

# ClearSkin Pro

## Your Ultimate Acne Solution



Er:Glass  
1540nm

The ClearSkin Pro applicator offers cutting-edge, non-ablative laser technology that delivers remarkable results with minimal discomfort. This advanced solution specifically targets acne concerns, revitalizes the complexion, and reveals radiant skin. The ClearSkin Pro 1540nm effectively treats all forms of acne vulgaris by targeting the sebaceous glands to reduce oil production, thus decreasing bacterial proliferation. Its fractional technology stimulates collagen production and promotes skin regeneration, improving the appearance of acne scars over time. Additionally, it enhances skin texture and tone, resulting in smoother, more youthful-looking skin. The treatment utilizes contact cooling for a pain-free experience, ensuring patient comfort throughout the procedure. Embrace ClearSkin Pro for your patients' best skin yet, and elevate your clinic's offerings with this FDA-cleared solution.



Courtesy of Pina Panchal MD, USA

Indications:



**Acne scars**



**Skin rejuvenation**



**Acne vulgaris**



**Stretch marks**



**Benign lesions**



**Xanthelasma**

Before



After 2 Tx

Courtesy of Dr. Ganesh Pai, India

Before



After 3 Tx

Courtesy of Platinum Laser Clinic, UK

# ClearLift™

## Advanced Laser Rejuvenation



Nd:YAG  
1064nm / 532nm

Alma's award-winning ClearLift applicator is the most comprehensive Q-Switched Nd:YAG treatment solution available today. This single laser module addresses multiple indications for all skin types with maximal power for optimal results. ClearLift delivers intense nanosecond pulses to the targeted area, creating a photo-acoustic effect that effectively removes tattoos and hyperpigmentation. Additionally, the pixel tip can be used to create controlled dermal wounds, stimulating collagen production for advanced skin rejuvenation treatments.

ClearLift features advancements with 5 interchangeable focal depths, allowing clinicians to precisely target both superficial and dermal pigment. This ensures accurate and effective treatment for various skin conditions, suitable for all skin types. ClearLift is globally renowned for achieving outstanding clinical outcomes and is often referred to as a 'lunchtime procedure' due to its quick, easy, and virtually painless application.



Before

Courtesy of Dr Kee Lee Tan, Rejuvenate Clinic, East Perth

After 3 Tx

Indications:



**Benign pigmented lesions**



**Photoaging**



**Skin rejuvenation**



**Skin laxity**



**Tattoo removal**



**Fine lines**



**Pigmentation**



**Enlarged pores**



**Photo damage**



**Melasma**

Before



After 1 Tx

Courtesy of Dr Kee Lee Tan, Rejuvenate Clinic, East Perth

Before



After 4 Tx

Courtesy of Erin K Larson, USA

# Dye VL / Dye SR

## Light Years Away The Alma AFT Difference



Harmony  
500-650nm

Experience a breakthrough in treating epidermal vascular and pigmented lesions with Alma's Dye VL/SR. Utilising Advanced Fluence Technology (AFT), this sophisticated light-based technology diffuses and corrects superficial redness and pigmentation, harnessing the transformative power of light energy. Alma's proprietary AFT technology features a narrow band spectrum, specifically designed for optimal haemoglobin absorption, making it highly effective for targeting superficial vascular lesions. Additionally, its high melanin absorption capability ensures efficient removal of spots and superficial pigmentation.



Before

Courtesy of Pietro Simone Clinic, London

After 5 Tx



Indications:



**Superficial pigmented lesions**



**Superficial vascular lesions**



**Skin rejuvenation**



**Angiomas**



**Rosacea**



**Telangiectasia**



**Port wine stains**



**Poikiloderma of Civatte**



Before



After 3 Tx

Courtesy of Dr. Alyona Kaznacheieva, Ukraine



Before



After 4 Tx

Courtesy of DD Clinical, UK

# NIR Face & Body

## Unique Near Infrared Skin Tightening Treatment



This advanced module uses a high-power light source in the near infrared spectrum to achieve safe and effective dermal heating, imparting a youthful glow to the skin. The pleasant treatment elevates dermal temperature, generating thermal injury to the tissue, which contracts existing fibers and stimulates the formation of new collagen. This process improves collagen alignment and thickness, resulting in tighter, brighter, and more youthful-looking skin. Unveil the secret to youthful skin with Alma's NIR technology, providing your patients with instant skin tightening and brightening while ensuring a safe and comfortable experience.



Before

Courtesy of Alma Lasers Clinical Department

After 2 Tx

Indications:



**Fine lines**



**Under eye rejuvenation**



**Wrinkles**



**Temporary relief for muscle joint pain**



**Skin laxity**



Before

Courtesy of Alma Lasers Clinical Department

After 4 Tx



Before

Courtesy of Courthouse Clinics, UK

After 3 Tx

# iPixel

It's time to say hello to  
**Radiant Confidence**



Transform your clinic's skin resurfacing capabilities with the Pixel Er:YAG applicator. This advanced applicator uses a 2940nm wavelength, which has a high absorption in water, making it the ideal technology for skin resurfacing treatments and benign lesion removal. The applicator provides highly effective fractional ablative laser skin resurfacing, significantly improving overall skin tone, elasticity, and texture, while reducing superficial pigmentation and pore size. Its innovative treatment mechanism promotes accelerated re-epithelialisation, allowing for faster healing and tissue regeneration.



Courtesy of Dr. Ganesh Pai, India

Indications:



**Scars**



**Verruca vulgaris (wart)**



**Superficial skin lesions**



**Skin resurfacing**



**Keratosis**



**Rejuvenation**



**Fine lines**



**Scar revision**



**Enlarged pores**



**Wrinkles**



**Debulking cysts**



**Acne scars**



**Epidermal nevi**



Before



After 1 Tx

Courtesy of Alma Lasers Clinical Department



Before

After 2 Tx

Courtesy of Shadi Danin Group of Clinics, UK

# LongPulse

An advanced treatment for  
**Vascular Lesions & Spider-Veins**



Nd:YAG  
1064nm

This powerful applicator utilises a 1064nm wavelength, providing greater depth of penetration and extended pulse duration to effectively treat deeper vascular lesions that shorter wavelengths cannot target. The applicator's unique open-angle view with an aiming beam significantly improves the visualisation of blood vessels beneath the skin. Additionally, simultaneous contact cooling technology minimises side effects and enhances patient comfort by cooling the skin during treatment. With high absorption by haemoglobin and deep penetration, the 1064nm LongPulse laser is highly effective for treating vascular lesions on both the face and body. It addresses superficial and deeper blood vessels, including leg veins, offering advanced skin rejuvenation and effective solutions for patients with vascular concerns.



Before

After 4 Tx

Courtesy of Dr BCK Patel, USA

Indications:



**Benign vascular lesions**



**Rosacea**



**Port wine stains**



**Telangiectasia**



**Hemangiomas**



**Reticular leg veins**



**Spider veins**



**Pseudofolliculitis barbae**



Before

After 1 Tx

Courtesy of Ass.Msc. Krenar Dobrosi ESTETHICA Clinic Pristina, Kosovo



Before

After 1 Tx

Courtesy of Phlebocosmetology Clinic, Russia

# Speed AFT

## Super Hair Removal (SHR)



Harmony  
**700-950nm**

Alma's SHR technology gradually heats the dermis to a temperature that effectively damages hair follicles and prevents regrowth while avoiding injury to surrounding tissue. The innovative sweeping In-Motion™ technique involves moving the applicator repeatedly over the treatment area, ensuring homogeneous energy coverage and significantly improving results. With a large spot size of 6.4 cm<sup>2</sup>, this broadband IPL applicator optimally targets melanin, resulting in safe and effective hair removal. Alma's SHR technology with Speed AFT offers a fast, efficient, and comfortable hair removal solution, making it an exceptional addition to your clinic's treatment offerings.

Indications:



**Hair removal**

Before



After 3 Tx

Courtesy of Alma Lasers Clinical Department



# Alma Lasers featured in

REVENGE  
**BODY**  
WITH  
KHLOÉ KARDASHIAN

N SERIES  
SKIN DECISION  
BEFORE & AFTER

THE **Doctors**

the **Real Housewives**  
OF ORANGE COUNTY

The **Guardian**

SPA+CLINIC

Dermatology  
TIMES

Aesthetic  
MEDICAL PRACTITIONER

marie claire



One Device,  
**Infinite Possibilities**

# Harmony Combinations

## Total Rejuvenation



### ClearLift & ClearSkin Pro

- Photodamage/Photoageing
- Collagen remodelling
- Fine lines
- Skin laxity
- Melasma
- Skin rejuvenation
- Mild acne scars
- Acne vulgaris

## Tone Enhancement



### ClearLift & Dye VL/SR

- Photodamage/Photoageing
- Collagen remodelling
- Fine lines
- Skin rejuvenation
- Superficial pigmented lesions
- Superficial vascular lesions

## Rejuvenated Glow



### ClearLift & NIR

- Photodamage/Photoageing
- Collagen remodelling
- Fine lines
- Skin laxity
- Skin rejuvenation

## Tone & Texture



### ClearLift & iPixel

- Photodamage/Photoageing
- Collagen remodelling
- Fine lines
- Skin laxity
- Melasma
- Skin rejuvenation
- Mild acne scars

## Solar Solution



### ClearLift & Cooled LongPulse

- Benign vascular lesions
- Globalised redness
- Photodamage/Photoageing
- Collagen remodelling
- Fine lines
- Skin rejuvenation

## Complete Vascular



### Dye VL/SR & Cooled LongPulse

- Benign vascular lesions
- Globalised redness
- Rosacea
- Port wine stains
- Spider-veins
- Telangiectasia
- Reticular leg veins

Focus.  
Flexibility.  
Freedom.

**That's Harmony.**



Alma **Harmony**  
XL PRO (SE)



Alma **Harmony**

**Alma**

**Alma Lasers Australia**

G05, 241A O'Riordan Street, Mascot NSW 2020  
Tel. 1300 511 862  
office@alma-lasers.com.au  
www.alma-lasers.com.au



Facebook



Instagram



WeChat



Little Red Book