

INSTRUCTIONS FOR USE:

GreenMed[®] GreenGuard Recycled Tray Corner Protectors



GreenMed[®] GreenGuard Tray Corner Protectors are intended for use with surgical trays to prevent the tearing of the sterilization wrap and to protect the corners of the trays from damage. These corners are made from 100% recycled polypropylene and colours may vary.

CONTRAINDICATIONS:

There are no known contraindications and/or adverse effects associated with the corner protectors.

INSTRUCTIONS FOR USE:

The corner protectors do not need to be cleaned or disinfected prior to use.

1. Place the corner protectors on the top and bottom corners of the tray before wrapping with sterilization wrap. Once in place, wrap the tray according to standard wrapping procedures.
2. The corner protectors can be sterilized according to the parameters detailed here.

STERILISATION INSTRUCTIONS:

Cycle Type	Temperature	Minimum Time - Full Cycle
Pre-Vacuum Gravity	134°C (270° F)	4 minutes
Gravity	121°C (250° F)	30 minutes

DISPOSAL:

The corner protectors are designed for one-time use and should be disposed of after use. These are 100% recyclable polypropylene #5 and should always be placed in a recycling receptacle where applicable or collected as part of the GreenMed[®] Keep Me in the Loop[®] program.

MANUFACTURED BY:



SUSTAINABLE
PLASTICS
AUSTRALIA

4015 HUON HIGHWAY
CASTLE FORBES BAY
TASMANIA, 7116 AUSTRALIA



SINGLE USE ONLY
DO NOT RE-USE

0420458837
info@greenmedical.com.au
www.greenmedical.com.au

