


# Hospital bed MAINTENANCE LIFT




The LT600HBML Hospital bed maintenance lift is designed to fit all hospital beds including bariatric beds weighing up to 600 kg.

Powered by an electric hydraulic lift, its remote control enables easy access for hospital maintenance or engineering staff to carry out servicing or maintenance repairs safely.

To prevent hospital beds from sliding off, the hospital bed maintenance lift is supplied with a 20 mm safety lip around the sides, as well as a Width Extension Kit (pictured below) for an additional 600 mm.

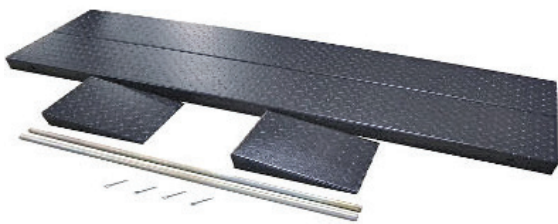

The Width Extension Kit.

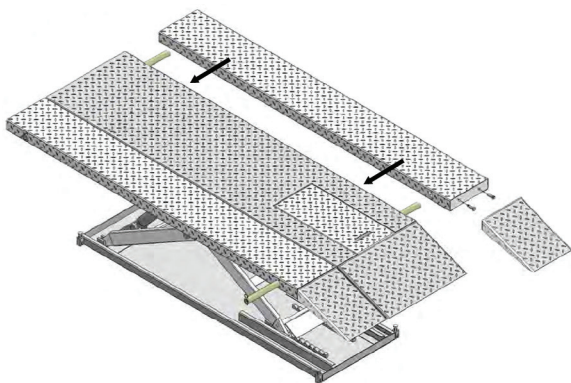

An example of Hospital Bed Maintenance Lift with the Width Extension Kit, adding 300 mm to each side.

Please note: This is an imported product.

## Typical applications

Suitable for lifting hospital beds for servicing and maintenance.

## Features

<b>Lift capacity</b>	Up to 600 kg
<b>Lifting time</b>	23 seconds
<b>Lowered height</b>	155 mm
<b>Raised height</b>	1090 mm
<b>Overall length</b>	2755 mm
<b>Ramp length</b>	400 mm
<b>Overall width</b>	1350 mm including the 600 mm Width Extension Kit.
<b>Deck type</b>	Chequer plate
<b>Usability</b>	<ul style="list-style-type: none"><li>• Expandable table configuration.</li><li>• Side table extensions.</li><li>• Add-on front deck to accommodate custom length beds.</li><li>• Electric hydraulic lift remote control.</li><li>• Hydraulic direct-drive cylinders, designed and made on ANSI standard, utilising oil seal in cylinder.</li><li>• Self-lubricating UHMW polyethylene sliders and bronze bush.</li></ul>
<b>Shipping weight</b>	290 kg
<b>Motor</b>	1.5 HP
<b>Power</b>	240V mains power operated

## Safety features

- Anti-slip/non-skid chequer plate platform.
- Automatic safety release system.
- Safety lip (20 mm) on three sides of platform (excluding ramp end) to prevent hospital beds from falling off.

## Note

The Hospital Bed Maintenance Lift is supplied partly flat-packed, some assembly required. Upon receipt of the unit, you will need to attach the hydraulic hoses and fill the reservoir with hydraulic oil (supplied with the unit) and bolt together the platform width extension. It is advisable that the unit is fastened to the floor before use. Full instructions are supplied with the unit.


## ORDER CODE

Hospital bed maintenance lift

LT600HBML

# Hospital bed MAINTENANCE LIFT

## Hospital Bed Maintenance Lift dimensions

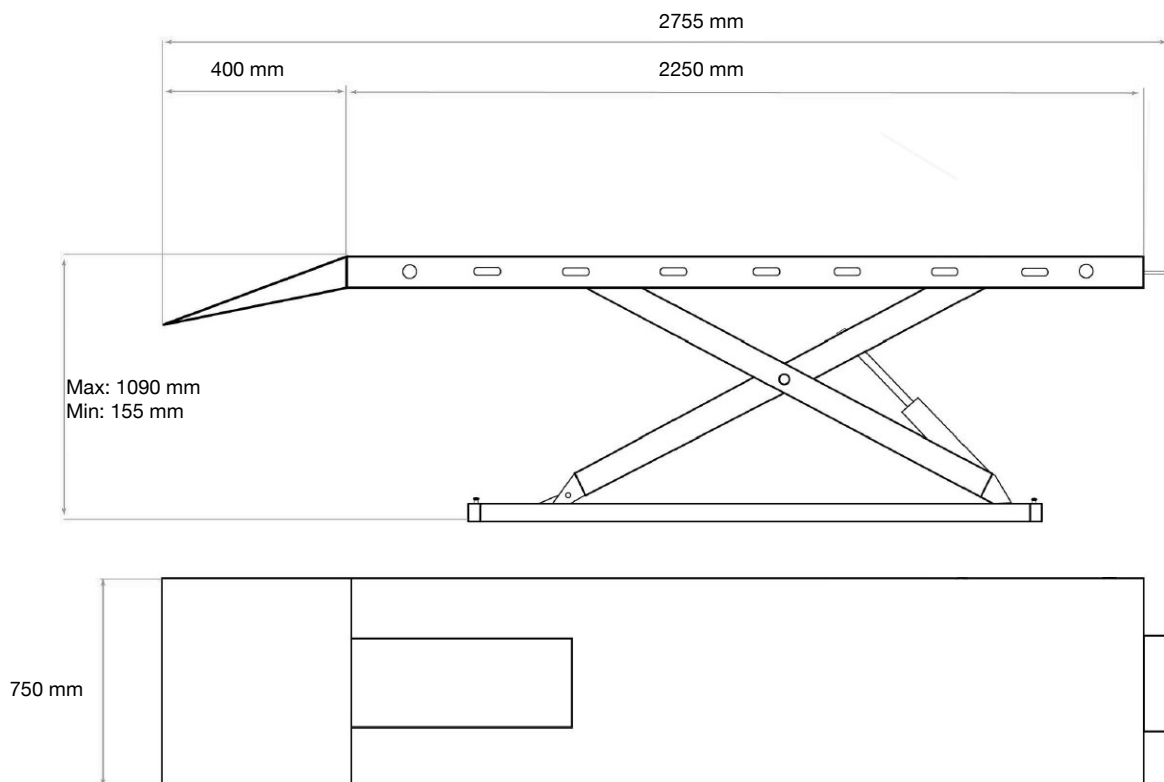

## Hospital Bed Maintenance Lift platform dimensions with the WEK (width extension kit).

