



Sterin-Trace

The Complete Sterilizer Record and Tracking System

The Sterin-Trace Sterilizer Record & Tracking System is designed to provide a simple method of record keeping & "Patient to Process" traceability of devices & instruments in the modern office based practice.

Sterin-Trace uses a simple intuitive check list that creates a reliable audit trail & enables the operator to record:

1. Sterilizer cycles
2. Process Expiry Date
3. Steam Penetration Test (Bowie-Dick)
4. DailyTest
5. Registered Authorised Operator
6. Patient
7. Maintenance Records

"IF IT'S NOT RECORDED THEN IT IS NOT DONE"

- Record keeping is the evidence that the practice has performed & monitored all stages of the sterilization process.
- To comply with, AS/NZS 4187, AS/NZS 4815 & ISO 13683 the practice must maintain "Patient to Process" tracking of instruments & devices.
- Responsibility lies with the practice management to ensure all the cleaning & sterilization processes are performed & recorded.

The implementation of the Sterin-Trace Quality System within the practice will:

- Provide a legal reference should a challenge occur.
- Reduce the risk to patients.
- Minimize the chance of nonsterile items being used.



Majac Medical Products Pty Ltd
Ph: +617 3265 6355
Fax: +617 3865 2729
www.majacmedical.com.au



Sterin-Trace

The Complete Sterilizer Record and Tracking System

CYCLE BY CYCLE RECORD - BATCHING


Batching will link your sterile product from the process to the patient. This is easily achieved by using twin layer “piggy back” process labels. This process label is attached to the item or pack, then peeled off & placed onto the patient notes or card when used. An extra label should be processed & placed on the sterilizer record sheet.

Individual operators should have an ID number. Where a practice has multiple sterilizers then each sterilizer should have its own ID number which is linked to all daily validations.


THE STERIN-TRACE PROCESS LABEL

The Sterin-Trace Label acts as an indicator (turns from Violet to Green), identifying items that have been through the steam sterilization process.

The Label will show: • Sterilizer Number • Cycle Number • Process Date • Expiry date • Operator

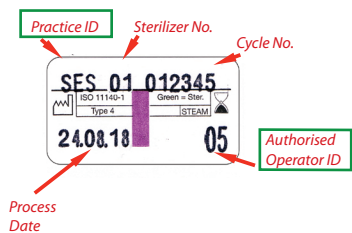

Changes from
Violet to Green

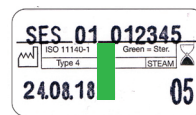

Sterin-Trace “Piggy Back” Process Label with Steam Indicator

Code: 602.401.9000/12 : Box of 12 rolls (9000)

Code: 602.401.9000/3 : Box of 3 rolls (each 750)

Colours available: Blue, Red, Green, Yellow, Orange, Grey

Sharpie Industrial Marker Pen
Specially formulated ink for use on
instruments & sterile packs
Code: S13801

PRINT OUTS - RETRIEVAL

There is no point keeping rolls of Autoclave “print out” streamers in a manner that prevents prompt retrieval. Using the Sterin-Trace System each sterilizer cycle “print out” is attached to the back page of the appropriate section of the Sterin-Trace Sterilizer Record System.

Two lines ... but it carries all information



Sterin-Trace 2 Line Label Applica- tor

Code: 601.002.0001

The Sterintech two line label gun is reliable, easy to use and durable. It will serve for years of intensive usage. Sterintech double labels for two line label guns are available with all type of sterilization indicators as well as a variety of colors. Colored labels are used to assist FIFO systems as well as easy recognition of sterile goods for a certain department.

Often the combination of a double label gun label and normal tape is a more cost-effective solution than steam tapes only. Next to that they offer better readable data and prevent writing errors.

Characteristics:

- ◆ Process Indicators (Steam, FO, EtO) conform to ISO 11140 - 1 class 1
- ◆ Clear transition of indicator into its end-color
- ◆ Easy to use label gun
- ◆ Double labels serve for an easy traceability and documentation
- ◆ Non-Toxic / Lead Free indicators
- ◆ Latex free label materials
- ◆ Can be fully recycled

Record keeping is the evidence that the practice has performed & monitored all stages of the sterilization process