

ICS PACIFIC VFT-28 Vaccine Fridge Digital Thermometer

Operating instructions

Functions

Current temperature display inside fridge (1)
Minimum temperature display (2)
Maximum temperature display (3)

Min temperature is factory pre-set at +2°C
Max temperature is factory pre-set at +8°C

Temperature accuracy +/-1°C



If the temperature reaches +2°C the alarm will immediately sound and the alarm ON symbol will flash

If the temperature reaches +8°C the alarm will immediately sound and the alarm ON symbol will flash

To silence the alarm press (6)

First time set up

The vaccine fridge must be operating between +2°C and +8°C

DO NOT turn on the thermometer yet

Connect the air temperature probe OR the vaccine temperature probe to the thermometer – the thermometer can be placed on or inside the fridge

Place the probe inside the fridge

Wait approximately ½ hour before turning on the thermometer

Press the ON/OFF button (4) once to turn on the thermometer

Normal operation

Min (2) / max (3) temperatures will remain on display until the Min-Max Reset button (5) is pressed

To reset min/max temperatures displayed press Min-Max Reset button (5)

Battery replacement (in approx. 12 months)

Remove the battery cover located on the back. Remove old batteries and replace with 2 new AAA batteries. Replace the cover.



Contact ENLAKE today on 1800 094 295

See our full range of products at www.enlake.com.au