



Ampoule cuts are painful and can cost you time and money but there is a better way.

Healthcare workers in the US alone report between 600,000 and 1 million sharps injuries per year. Many more go unreported. Over one quarter of sharps injuries occur while opening glass ampoules.

Here at Qlicksmart we have been on the job developing the best ampoule opener on the market, so now ampoule opening is as easy as 1, 2, 3...

1. INSERT the ampoule lid into the SnapIT.
2. SNAP open the ampoule.
3. EJECT the ampoule lid directly into a sharps bin.

Qlicksmart SnapIT is a simply safer way to open glass ampoules. These easy-to-use multi-use ampoule openers avoid the unacceptable, painful and dangerous sharps injuries seen when ampoules are opened by hand.

...plus, SnapIT really is easier than using your fingers!

How do I protect my colleagues and myself from injury and risk?



CONTACT US NOW
for more information



QLICKSMART PTY LTD
Phone: +61 7 3844 1182 Fax: +61 7 3844 1183
Level 1, 148 Boundary Street
West End, Brisbane, Qld 4101, Australia
www.qlicksmart.com mjs@qlicksmart.com

SnapIT
Multi-Use | Ampoule Opener



A simply safer way to open glass ampoules.

Protect yourself from the risk of contamination and sharps injury.



Better and SAFER than fingers!

PERSONAL Edition

The Personal SnapIT is a great size for personal use. Its compact and stylish design means you can carry it with you, allowing for easy and safe ampoule opening of most standard ampoules.

- prestigious look and feel.
- anodised aluminium in two colour choices.
- great gift for a friend or for yourself.
- quality promotional item.
- fasten to a lanyard, carry in your pocket.
- autoclavable if required.
- personal SnapIT opens 1ml to 15ml ampoules.
- opens 70% of all ampoules.



SnapIT works, and it works *well*.
If you've tried other ampoule openers without success, be sure to try this excellent product!

COMMON FEATURES of all SnapIT:

- complies to WPHS Standards.
- protects yourself and your staff from sharps injury.
- contains the lid until you are ready to eject.
- reduces the likelihood of smashed ampoule lids.
- in the unlikely event of smashed ampoule lid, shards are contained within the SnapIT.
- easy to clean.
- can be branded with your company logo.

Choose a SnapIT for the purpose you require!

TROLLEY Edition

The Trolley Edition SnapIT is perfect for the hospital environment, chain a set of trolley SnapIT's where ampoules get opened in your facility, and get serious about staff safety!

- quality anodised aluminium.
- flat edge to stop rolling.
- anchor to a location using light chain.
- autoclavable if required.
- regular opens 1ml to 15ml ampoules.
- large opens 5ml to 25ml ampoules.

THE FLAT EDGE IS A FEATURE ON ALL TROLLEY EDITION.



LITE Edition

SnapIT Lite comes in regular and large. The lightweight plastic components means you can add this to your lanyard with other items and not be weighed down.

- made from sturdy, high strength plastic.
- hexagonal anti-roll collar.
- great promotional item or gift.
- regular 1ml to 15ml ampoules, that's 70% of all ampoules used.
- large opens 5ml to 25ml ampoules.

HEXAGONAL COLLAR HELPS STOP ROLLING.



Who should use SnapIT?

- Hospitals
- Nurses
- Paramedics
- Anaesthetists
- Educational facilities
- Veterinary practices
- Dentist surgeries
- General practices
- Anyone who opens glass ampoules.

Patients at home particularly struggle with opening of glass ampoules. SnapIT is perfect for this application.