

MELAG Autoclave



Wipe down black plastic rim on door and inside chamber weekly with a lint free towel and distilled water

On/Off button for Autoclave

Empty dirt water tank always before filling top clean tank

- Vacuum Test must be completed first in the morning on a cold chamber. Change program to vacuum test then press start Once complete please check program is back to 134 degree wrapped.
- Always remember to empty the dirty water tank before re filling with distilled water.
- Turn off machine at the bottom at the end of the day.

BEPDW5LRT
BEPDW20LTR
SPIPAP510
DHTEC09

DISTILLED WATER 4L T.
DISTILLED WATER 20 L TR.
MELAG AUTOCLAVE PRINTER PAPER ROLL 57 MM.
PRINTER RIBBON ERC-09 AUTO PAPER.