

TEMPSTRIP

- Hold the **TEMPSTRIP** by both ends and press against on dry forehead for 15 seconds.
- Read **TEMPSTRIP** while it is on the forehead.
Green indicates correct temperature
Tan / red = deduct 0.5°C (0.9°F)
Blue = plus 0.5°C (0.9°F)
- Use **TEMPSTRIP** at room temperature 22 ~ 25°C (72 ~ 77°F)
- Do not measure temperature if you have been eating, drinking or exercising within at least 30 minutes, or if you just come in from outdoors.
- **TEMPSTRIP** should be cleaned with a soft cloth and stored in its sleeve to keep away from sunlight and heat source.
- Consult doctor when you think you are fever or when you are in doubt.

IRtek International - P.O. BOX 435, Joondalup, W.A. 6919, Australia
sales@irtek-temp.com / irtekaustralia@gmail.com

© IRtek International. All rights reserved. Specifications subject to change without prior notice.