

Your Need ... Our Innovation®

**ONLY ONE** 

## Flow-Safe II<sup>®</sup> Disposable CPAP Therapy System

FLOWSAFE ]

- Ideal for wherever CPAP therapy is appropriate
- Pre-hospital or in hospital use
- Compact and easy to store no extra parts
- Color-coded manometer verifies delivered **CPAP** pressure
- Uses 50% Less Oxygen Consumption than the original Flow-Safe
- Comfortable head harness with quick disconnect clips for easy access to patient's face
- Forehead support reduces pressure on the nose and easily adjusts
- Easy Set up & Training just attach to standard flowmeters/Flow determines CPAP pressure. Yellow Flow/CPAP chart on all devices illustrates flow rate with corresponding delivered CPAP pressures.

#### CONNECT TO FLOW SOURCE ONLY FLGW-SAFE II

CPAP/PEEP (cm H <sub>2</sub> O)	Flow (LPM)
5.0	8-9
7.5	10 - 12
10.0	13 - 14
13.0 (MAX)	FLUSH

### www.mercurymed.com







P O Box 2541 Southport, QLD 4215 Ph: 1300-786-454 F: 5591 8522 E; sales@questsurgical.com.au ABN 33 118 090 618 ACN 118 090 618

Mercury Medical<sup>®</sup>

Your Need ... Our Innovation®

# ONLY FLOMSAFEII® DISPOSABLE CPAP SYSTEM

Flow-Safe II<sup>®</sup> Disposable CPAP Therapy System Compared to Non-invasive Mechanical Ventilation



Contents lists available at ScienceDirect American Journal of Emergency Medicine

journal homepage: www.elsevier.com/locate/ajem



Is the flow-safe disposable continuous positive airway pressure (CPAP) system as effective as non-invasive mechanical ventilation (NIMV) in the treatment of acute cardiogenic pulmonary Oedema?

Ilhan UZ, MD<sup>4,\*</sup>, Güçlü Selahattin KIYAN, MD<sup>4</sup>, Enver ÖZÇETE, MD<sup>4</sup>, Sercan YALÇINLI, MD<sup>4</sup>, Mehmet Birkan KORCAN, MD<sup>4</sup>, Yusuf Ali ALTUNCI, MD<sup>4</sup>, Murat ERSEL, MD<sup>4</sup>, Funda Karbek AKARCA, MD<sup>4</sup>, Oğuz YAVUZCİL, MD<sup>b</sup>

\* Department Of Emergency Medicine, Ege University Faculty of Medicine, Izmir, Turkey b Department Of Cardiology, Ege University Faculty of Medicine, Izmir, Turkey

According to this recent American Journal of Emergency Medicine, For acute cardiogenic pulmonary oedema, (ACPO) patients, the Flow-Safe<sup>®</sup> Disposable CPAP device is "as effective as NIVM, in improving blood pressure, pulse, respiratory rate and blood gas."

Flow-Safe II, Flow-Safe II  $\text{EZ}^{\otimes}$  and Flow-Safe II+ $^{\otimes}$  can be solutions for situations where non-invasive mechanical ventilation equipment is scarce or unavailable.

ORDERING INFORMATION			
Part #	Description	Packaged	
#10-57209	Flow-Safe II CPAP System w/ large adult deluxe mask w/ straight swivel port, head harness	5/Box	
#10-57210	Flow-Safe II CPAP System w/ medium/small adult deluxe mask w/ straight swivel port, head harness	5/Box	
#10-57211	Flow-Safe II CPAP System w/ child deluxe mask w/ straight swivel port, head harness	5/Box	

\* Other configurations available

© 5/2020



### www.mercurymed.com

ML2020-424



Since 1963



P O Box 2541 Southport, QLD 4215 Ph: 1300-786-454 F: 5591 8522 E; sales@questsurgical.com.au ABN 33 118 090 618 ACN 118 090 618