

# Feel the freedom

With a portable oxygen concentrator



# Understanding oxygen therapy

If you, or a person you care for, has been prescribed supplemental oxygen therapy, BOC can help you select the best solution to suit your needs.

BOC supplies a range of lightweight portable oxygen concentrators ideal for people who are on the move or travelling. These battery-operated models provide you with freedom of movement to enjoy life.

## Portable oxygen concentrators

BOC's portable oxygen concentrators come in continuous or pulse dose models.

Pulse dose models release a bolus of oxygen when they sense inhalation and have a range of settings to control the size of the bolus. These setting numbers are not equal to continuous flow rates, e.g. setting 2 does not mean 2L/min. It is important that you discuss with your care provider what setting to use your device at.

If you need to hire a specific model to take to your appointment for titration, talk to our customer service team.

## Flying with an oxygen concentrator

When you are onboard an aircraft, the altitude can cause your oxygen saturation to drop and you may need to use your device at a higher setting.

Discuss with your care provider what setting to use your device at while flying.

**We recommend you contact your airline in advance to seek their approval and book a seat with a power outlet.**



## Portable oxygen concentrators

BOC offers a range of portable oxygen concentrators with demonstrated quality and performance. The following table outlines the specifications of our portable devices.

If you need assistance in selecting the right product, consult your care provider or call our customer service team on 1800 050 999.



**Price match<sup>1</sup>**  
**Free delivery<sup>1</sup>**  
**Try before you buy<sup>2</sup>**

Model	Philips SimplyGo™	GCE Zen-O™	Inogen One™ G3™	Inogen One™ G5™	Caire FreeStyle™ Comfort™
Continuous flow	0.5-2 L/min	0.5-2 L/min	×	×	×
Pulse dose	Six settings	Six settings	Five settings	Six settings	Five settings
Recommended for use while sleeping	✓	✓	×	×	×
Weight kg	4.5 kg	4.7-5.6 <sup>3</sup> kg	2.2 - 2.8 <sup>3</sup> kg	2.3 - 2.8 <sup>3</sup> kg	2.3 - 2.7 <sup>3</sup> kg
Battery run time <sup>4</sup>	54 min continuous - 3 hrs pulse	45 min-1.5 <sup>3</sup> hrs continuous / 4-8 <sup>3</sup> hrs pulse	4.5 - 9.5 <sup>3</sup> hrs	5 - 10 <sup>3</sup> hrs	4 - 8 <sup>3</sup> hrs
Carrying accessories	Carry bag and trolley	Carry bag and trolley	Carry bag/ optional backpack	Carry bag/ optional backpack	Carry bag/ optional backpack
Warranty	<b>Device:</b> Two years <b>Sieve beds:</b> One year <b>Battery:</b> Three months	<b>Device:</b> Three years or 15,000 hrs <b>Sieve beds:</b> Three years or 15,000 hrs <b>Battery:</b> One year	<b>Device:</b> Five years <b>Sieve beds:</b> One year <b>Battery:</b> One year	<b>Device:</b> Five years <b>Sieve beds:</b> One year <b>Battery:</b> One year	<b>Device:</b> Three years <b>Sieve beds:</b> Three years <b>Battery:</b> One year
	<a href="#">View online</a>	<a href="#">View online</a>	<a href="#">View online</a>	<a href="#">View online</a>	<a href="#">View online</a>

<sup>1</sup> For purchases only. Valid quote or a link to an Australian supplier online store is needed. Price match applies for the same product configuration.

<sup>2</sup> You may rent a specific model to try before you purchase. If you decide to proceed with a purchase post trial of the same model, you will be eligible for a discount off the purchase price.

<sup>3</sup> With extended battery

<sup>4</sup> At setting 2 Battery run times are based on average values and presented for comparison purpose. Actual run times may vary.



## Making portable oxygen easy

### Purchasing a portable oxygen concentrator

Purchasing a portable oxygen concentrator (POC) may appear to be a complex decision. We are at hand to help you every step of the way with all the information you need.

Experience a purchase with complete peace of mind with:

- Price match<sup>1</sup>
- Free delivery<sup>1</sup>
- Wide range of optional accessories and consumables
- First class customer service and after sales support
- Qualified workshops for all servicing needs
- Try before you buy and receive a discount off purchase price<sup>2</sup>

### Rent a portable oxygen concentrator

Whether you are going on a short trip or need a portable concentrator for a longer period, you can enjoy our selection of well maintained rental devices. To rent a POC:

-  In consultation with your doctor, complete our self-funded referral form which can be found at [www.boc.com.au/homecare](http://www.boc.com.au/homecare)
-  Email the above to [healthcare@voc.com](mailto:healthcare@voc.com)
-  Our customer service will call you within 24 hrs to discuss your options

We are proud to support veterans and accept DVA cards. Your referring doctor is able to access the referral form at the following web page:

[www.boc.com.au/homecare](http://www.boc.com.au/homecare)

- 1 For purchases only. Valid quote or a link to an Australian supplier online store is needed. Price match applies for the same product configuration.
- 2 You may rent a specific model to try before you purchase. 'If you decide to proceed with a purchase post trial of the same model, you will be eligible for a discount off the purchase price'.

---

## **BOC Limited**

Committed to your quality of care providing respiratory products with demonstrated effectiveness.

email: [healthcare@boc.com](mailto:healthcare@boc.com)

Phone: 1800 050 999

10 Julius Avenue,  
North Ryde, NSW 2113  
Australia

[www.boc.com.au/healthcare](http://www.boc.com.au/healthcare)



BOC is a trading name of BOC Limited, a subsidiary of Linde plc. © BOC Limited 2021. Reproduction without permission is strictly prohibited.

Details given in this document are believed to be correct at the time of printing. Whilst proper care has been taken in the preparation, no liability for injury or damage resulting from its improper use can be accepted.

Always read and refer to the user manual before operating equipment.