

The **ONLY SKIN RESURFACING**

System You Need *Now More Customizable Than Ever*

VENUS Viva™ MD is a highly effective and customizable professional skin resurfacing and skin tightening device, treating a range of mild to severe skin conditions.

The NanoFractional RF™ is an alternative to microneedling and fractional laser treatments. Treatments deliver exceptional results with low downtime and are safe for all skin types.

With our DiamondPolar™ applicator, employing our proprietary (MP)² technology for wrinkle reduction*, patients can comfortably achieve a firmer, more youthful looking appearance.


VENUS Viva™ MD can replace most other skin resurfacing devices with one lightweight, tabletop device. Experience a higher ROI, elevate patient satisfaction, and ensure greater profitability through offering a wide array of effective and in-demand treatments.



- **CUSTOMIZABLE SOLUTIONS FOR DIVERSE SKIN CONDITIONS:** Tailor power and pulse duration to treat various skin damage levels, from mild to severe. Clinical studies endorse NanoFractional RF™ technology for enhancing skin texture, reducing striae alba, and managing pigmentation issues.
- **CUTTING-EDGE TECHNOLOGY:** The VENUS Viva™ MD applicator utilizes NanoFractional RF™ for safe, effective, and comfortable treatment of diverse skin conditions. Equipped with SmartScan™ technology, it offers deeper energy penetration, flexible treatment options, and improved energy delivery for visible results.
- **SAFE FOR ALL SKIN TYPES*:** Treatments are suitable for all skin types, including Fitzpatrick Skin Types IV–VI. Clinical studies indicate significantly lower post-inflammatory hyperpigmentation (PIH) compared to other laser treatments, ensuring safety for diverse skin tones.
- **LOW DOWNTIME:** NanoFractional RF™ with SmartScan™ technology minimizes downtime by creating micro-dermal wounds surrounded by intact tissue, enabling faster recovery and reduced downtime.

RESULTS REVEALED

Results after **3 Skin Resurfacing** treatments

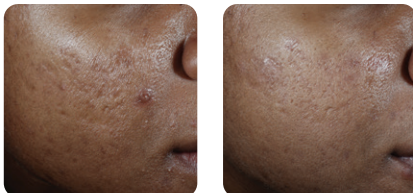

Courtesy of Neil Sadick, MD

Results after **3 Skin Resurfacing** treatments



Courtesy of Brow & Beauty

Results after **3 Wrinkle Reduction** treatments



Courtesy of Alpha Aesthetics

NANOFRACTIONAL RF™ with SMARTSCAN™

FEATURED IN THE VIVA™ MD APPLICATOR for procedures requiring ablation and resurfacing of the skin, customizable with 2 energy delivery options:



- **160 PIN TIP** – up to 62 mJ/ pin, for milder skin texture irregularities such as fine lines, dyschromia, and enlarged pore
- **80 PIN TIP** – up to 124 mJ/ pin, for more severe skin concerns such as acne scars and other scars, striae, and deep wrinkles
- Achieves therapeutic depth of impact with high energy density, distributed through a small footprint per pin (each 38 µm²).
- Leaves sufficient intact tissue in between for faster healing.
- Utilizes proprietary SmartScan™ technology that employs a unique algorithm to scan between randomized groups of four pins.
- Delivers energy at variable impact zone densities in a single tip, ensuring safe treatments for all skin types.
- SmartScan™ also uses pattern selection technology to create customized patterns for maximum flexibility and control during treatment.

(MP)² TECHNOLOGY

FEATURED IN THE DIAMONDPOLAR™ APPLICATOR for non-invasive wrinkle reduction*

- Our proprietary (MP)² is the synergistic combination of Multi-Polar Radio Frequency (RF) and Pulsed Electro Magnetic Fields (PEMF). Multi-Polar RF heats and directly stimulates fibroblasts, while PEMF is known to promote angiogenesis and induce fibroblast proliferation through release of the growth factor FGF-2, resulting in increased collagen synthesis.
- (MP)² is effective in collagen remodeling for skin tightening, wrinkle reduction and the creation of new capillaries for renewal of blood supply.
- With the capabilities of Multi-Polar RF applicators, VENUS Viva™ MD uses a complex algorithm to deliver homogeneous energy and volumetric heating to multiple tissue depths, allowing for quick and safe buildup of heat and easy maintenance of therapeutic temperature throughout the treatment.
- The unique electrode placement of the DiamondPolar™ applicator allows for treatment around the eyes, along the jowls, and other small areas to smoothen fine lines and wrinkles while providing firmer, more youthful looking skin.
- Treatments with this applicator are comfortable with no downtime, and RF technology is proven safe for all skin types.

NANOFRACTIONAL RF™ APPLICATOR

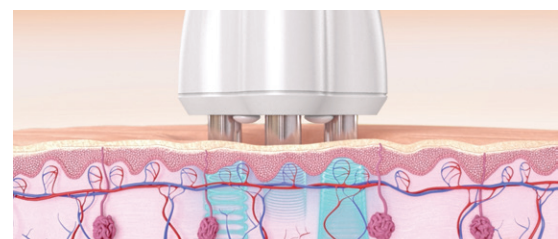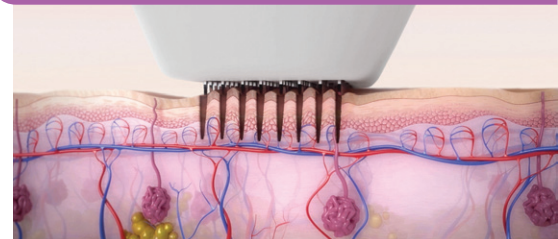

DIAMONDPOLAR™ APPLICATOR*

OUR COMMITMENT TO YOUR SUCCESS

CLINICAL EDUCATION

- World-class training & educational program
- Clinical training & support from highly skilled & experienced aesthetic specialists
- Virtual clinical support sessions
- 24/7 online access to clinical training

MARKETING SUPPORT

- Access to downloadable marketing materials to promote your new VENUS treatments in your practice
- Business listing on our Clinic Finder website
- Patient education tools & resources

PRACTICE DEVELOPMENT PROGRAM

- Led by a team of marketing experts
- Industry leading content to effectively launch, promote & grow your practice
- Proven Business & Marketing staff training series, strategies & resources

EASY UPGRADES

- Seamless access to the newest innovative technology
- Immediate upgrade anytime, including mid-contract
- Prevents technology obsolescence

📞 888-907-0115
 🌐 venus.ai
 ✉ info@venus.ai

📍 **Corporate Office:**
 235 Yorkland Blvd, Suite 900,
 Toronto, ON M2J 4Y8, Canada

📍 **USA Office:**
 4001 SW 47th Ave, Suite 206,
 Davie, FL 33314, United States