USED STAND-A-LONE IN A COMMMUNITY AREA and/or NURSE CALL SYSTEM

NO MORE CORDS
BETWEEN A MONITOR
AND A SENSOR MAT

MULTI PURPOSE ALARM MONITOR, POWERED ADAPTOR OR BATTERY

TRANSMITTER MODULE INCLUDED (RJ11 PORT)

REQUIRES BED OR CHAIR MAT WITH RJ11 PLUG

REDUCE THE RISK OF FALLS & POTENTIAL INJURIES

PORTABLE

EASY TO USE

PLUG & PLAY OPERATION

COST EFFECTIVE



DELUXE FALL ALARM MONITOR & WIRELESS TRANSMITTER KIT





PRODUCT CODE:

Part No: MED-TS2023K-W6T3

Kit includes: Monitor, Nurse Call Connection Lead – Tip/Sleeve (T3), Wall Mount Bracket, Power Pack, Silicon Strap and Transmitter Module.

Indigo Care's 8 Channel Falls Monitor & Transmitter is a perfect solution for monitoring your residents in a "Stand-A-Lone" community area and/or via a nurse call system. It is designed for our chair & bed mats/pads (with RJ11 plug) and a wireless module (RF Transmitter) with the monitor (RF Receiver).

A unique feature of the Alarm Monitor is its ability to program up to 8 sensor mats and transmitters giving you more monitoring options. It can be easily used with a wheelchair or in a communal area, from 1 to 8 chair sensor mats per monitor, with each mat requiring its own transmitter. Since the transmitter module to monitor unit is cordless, it allows your residents to be simultaneously monitored anywhere in the facility (recommended line-of-sight is 30 metre).

To set up, Indigo Care's bed or chair mat's RJ11 connection will plug into the Fall Monitor's Transmitter which is powered by 2 x AAA batteries. Connect your sensor mat to the Transmitter's RJ socket and switch the on/off switch to the ON position to program to the Fall Monitor. (separate programming instruction are provided).

We have included a wall mount bracket and the silicon chair mount strap which allow easy installation of the Monitor. The Monitor is powered either by its included power adaptor or 3 x AA batteries that you can purchase separately. Three different alarm tones or mute mode can be selected as required. Once a resident gets up from the sensor mat, it will alarm and alert the staff, through your nurse call system. That's it; easy to set up and easy to use!

Note: The kit includes a 6.3mm stereo lead with Tip & Sleeve (T3) wiring. Please check the wiring configuration of the Call Point before using the lead.

Note: Monitor & Transmitter are sold as a Kit. Sensor Mats can be purchased separately.

KEY SPECIFICATIONS:

- Nurse Call, DC Output & Battery Compartment
- Program up to 8 wireless Bed or Chair Mats RJ11 connection
- Alarming Monitor Visual Screen
- 1/2 metre cable with 6.3mm Stereo Jack Plug T3 (Tip/Sleeve)
- 3 Selectable Alarm tones
- 3 levels Volume Control: Low, Medium or High
- 3 Delay Settings: 0 sec, 2 sec or 15 min
- Reset/Pause Button Pause up to 10 minutes, cancel at any time
- Volume Control and Mute Button
- Lower Battery Warning
- · Plug Pack Disconnect Warning & Transmitter Offline Warning
- · Power On/Off Switch
- Monitor 12V Power Adaptor (included) or 3 x AA Batteries (included)
- Transmitter 2 x AAA Batteries (included)
- · Warranty: 1 year

INDIGO CARE

(Horizon Electronics Company, T/as) 2/93 Rushdale Street, Knoxfield, VIC, 3180 PH +61 3 9763-2988 www.horizone.com.au ABN 36007302801

ORDER INFORMATION
Email your purchase orders to sales@indigocare.com.au